Spis treści

CHIRURGIA / SURGERY

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
Fig. 1,7,13,17,18,22,33A,51A,67A,79

Berten Extraction forceps set contents
10 pieces

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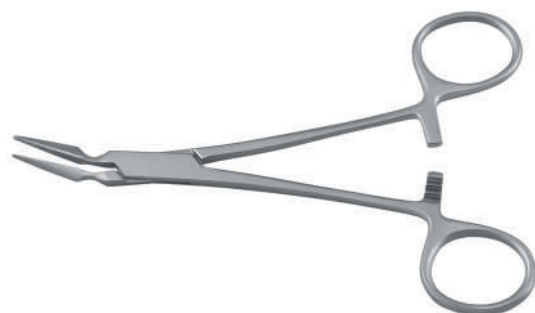Forceps for extraction of primary teeth

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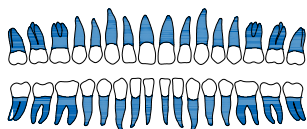
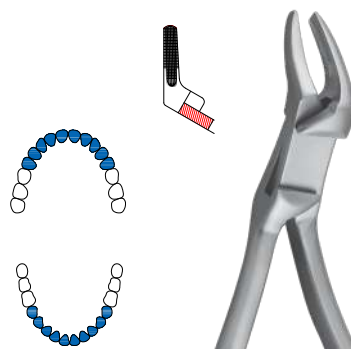
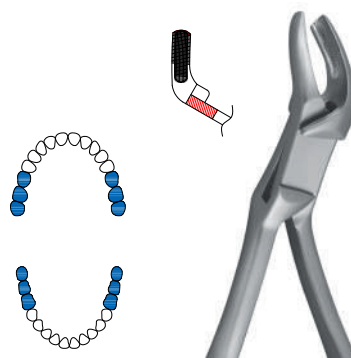
MD-100-031

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Meissner extraction
forceps set contents 3 pieces

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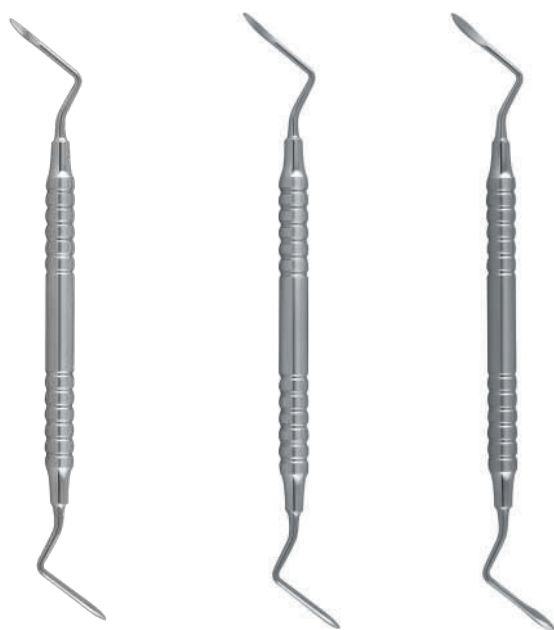
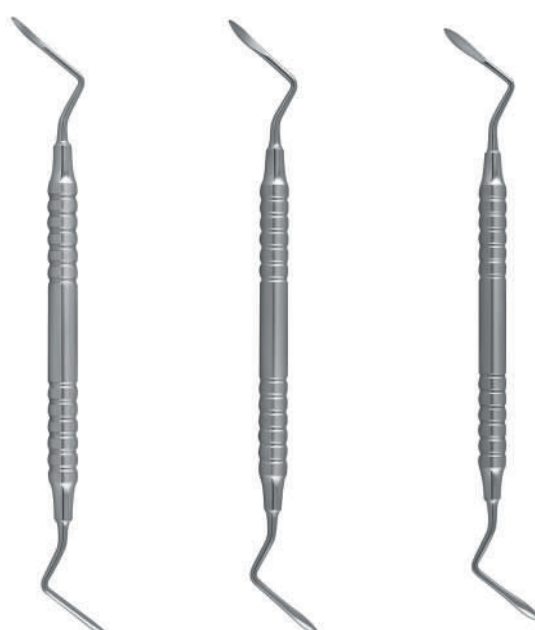
MD-300-001 | **MOLT ostro-ostry**
18 cm | sharp-sharp

MD-300-002 | **WILLIGER**
16 cm

MD-300-004 | **MICRO MEAD-WEST z nożem**
18 cm | with knife

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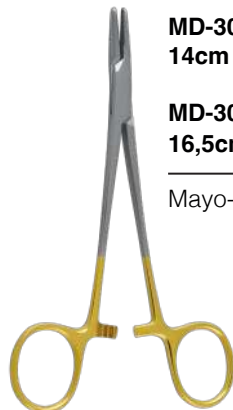
Dental syringes and intraligamental syringes

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dental syringe with
aspiration standard

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dental syringe with
aspiration breech loading

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dental syringe with
aspiration type 3 rings

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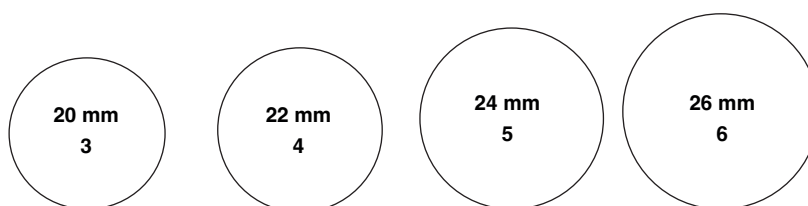
Diagnostic set no. 2 ergonomic handle

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Explorers, probes and mirrors handles

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Moulding instrument, ergonomic handle TiN*

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Cement spatula one-sided 8,5 mm

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Rubber dam forceps, punch, rubber dam frames

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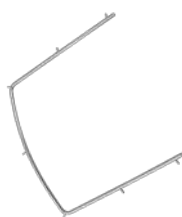
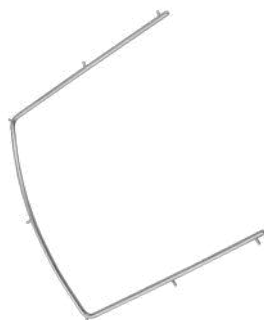
EN: STERILIZATION, CARE & MAINTENANCE INSTRUCTIONS

General Instructions:

You have bought a quality instrument. For the best results, please observe the following instructions.

The following instructions apply to all reusable medical devices supplied by Meditrans, unless otherwise stated on packaging of the product.

These instructions are intended for use only by persons with the required specialist knowledge and training.

Regulations:

These instructions offer general guidelines, in addition to these please observe national regulations and standards concerning medical instrument use.

For patients suffering from CJD, HIV, Hepatitis, or any other contagious diseases, please observe the relevant national regulations concerning the medical instrument reprocessing. Always strictly follow the instructions of cleaning agent, disinfectant, and equipment manufacturers.

The operator is solely responsible for the success of the selected procedure and validation of its effectiveness.

Warnings:

Follow instructions and warnings as issued by the manufacturers of any decontaminants, disinfectants, and cleaning agents. Wherever possible avoid use of mineral acids and harsh, abrasive agents.

No part of the process shall exceed 140°C.

Some sensitive materials (e.g., Aluminium) are damaged by highly alkaline solutions (pH >10).

Exercise particular caution when cleaning devices with long, narrow cannula, hinges, and blind holes.

Safety guidelines:

Special care must be taken to ensure personal safety,

The procedures may be performed only by fully trained personnel.

Protective clothing, gloves and eyewear must be worn in accordance with local health and safety standards.

Sharp instruments must be handled separately in baskets or cassettes,

Please strictly follow the Manufacturer's instructions and guidelines.

Reprocessing Limitations

Repeated processing can have a minimal effect on the instruments. The end of an instruments service life is normally determined by wear and damage in use.

New Instruments:

All instruments are supplied non-sterile unless otherwise stated on label.

Remove all packaging including plastic protection sleeves on instrument tips. Clean the new product either manually or mechanically prior to the initial sterilization.

Used Instruments:

Wherever possible, do not allow blood debris or bodily fluids to dry on instruments. For the best result and to prolong the life of the medical devices, reprocess immediately after use.

If they cannot be reprocessed immediately, use an enzymatic foam spray cleaner to prevent contaminants from drying.

Preparation for decontamination:

Reprocess all instruments as soon as it is reasonably practical following use. Disassemble only where intended, without the use of tools unless specifically provided by the manufacturer.

Where instructions for disassembly are required, these are available with the device.

Cleaning

1.1 Ultrasonic Cleaning

Use only validated and CE marked equipment.

Always follow the manufacturer's instructions for use.

Do not over-clean. A maximum immersion time of 10 minutes is recommended. Avoid over-crowding of instruments. Place heavier instruments at the bottom of the basket.

Handle 'sharp' tools (scissors, knives, chisels etc.) separately.

Keep joints open and ratchets unlocked.

Rinse instruments with water to remove the cleaning solution.

1.2 Automatic Washer /Disinfector

Use only CE marked and validated washer-disinfector machines and low-foaming, non-ionizing cleaning agents and detergents following the manufacturers' instructions for use, warnings, concentrations, and recommended cycles.

Load instruments carefully, with any box joints and hinges open and so that any fenestrations in instruments can drain.

Place heavy instruments with care at the bottom of containers, taking care not to overload wash baskets.

Place instruments with concave surfaces facing down to prevent pooling of water. Where available, use appropriate attachments to flush inside reamers and devices with lumens or cannula.

Ensure that soft, high purity water which is controlled for bacterial endotoxins is used in the final rinse stage.

Note: Automated cleaning may not be suitable for all lumens and cannula, in which case clean manually with a water jet gun, if available, and an appropriate brush (and stiletto if provided) that reaches the depth of the feature. After manually cleaning, pass all devices through an automatic cleaning cycle to achieve full disinfection.

Note: These instructions have been compliant with the standard when using a validated washer-disinfector cycle to include two cold rinses < 35°C, a detergent cycle and a rinse cycle both at >50°C, a disinfection cycle at a temperature of between 80°C and 87°C for a minimum holding time of 1 minute (actual holding time in excess of 2 minutes 50 seconds) and a 20 minute drying cycle. The detergent used was a low foaming, non-ionising spray wash detergent cleaner (max 12 pH; and the rinse aid a neutral pH low foaming, non-ionic surfactant with isopropyl alcohol.

1.3 Manual Cleaning

Use a double sink system (wash/rinse) dedicated for medical instrument cleaning (not used for hand washing),

In the first sink (wash) keeping the instrument submerged, using an autoclavable brush, apply CE marked cleaning solution to all surfaces until all contaminants have been removed.

In the second sink (rinse), rinse instruments thoroughly using soft, high purity water, which is controlled for bacterial endotoxins, so that the water reaches all parts of the instruments, then carefully dry or use a drying cabinet.

Ensure that the water temperature does not exceed 35°C

Always brush away from the body and avoid splashing.

Pay particular attention to serrations, knurling, hinges, ratchets, and tubes.

Ensure hinged instruments are thoroughly cleaned in both open and dosed position.

Clean delicate instruments separately to avoid damage.

Do not use steel wool or steel wire brushes.

Use only CE Marked detergents with a neutral pH (7).

Note: Manual cleaning is NOT a disinfection process.

Inspection and Maintenance

After cleaning, visually inspect all surfaces, cannulation, ratchets, joints, holes, and lumens for complete removal of soil and fluids, if ANY soil or fluid is still visible, return the instrument for repeat decontamination.

Lubrication: Apply surgical grade lubricants to hinges, joints and moving parts as per the lubricant manufacturer's instructions. Use only non-industrial type, silicon water-soluble lubricant, not industrial type.

Visually inspect and check: all instruments for damage and wear: cutting edges must be free of nicks and present a continuous edge; jaws and teeth align correctly; all articulated instruments have a smooth movement without excess play; locking mechanisms (such as ratchets) fasten securely and close easily, long, slender instruments are not distorted; any component parts fit and assemble correctly with mating components. Remove any blunt, worn out, flaking, fractured or damaged instruments for repair or replacement.

Note: if any instrument is returned to the manufacturer / supplier, the instrument must be decontaminated and sterilized and be accompanied by the relevant documented evidence.

Packaging: All instruments to be packed according to the method of used following national and international standards.

Cold Disinfection:

Use only CE marked products.

Do not use bleach (sodium hypochlorite).

Follow all of the manufacturer's instructions and warnings. After disinfection, rinse thoroughly under running water. Note: Disinfection is not an alternative to Sterilization

3. Sterilization

3.1 Steam Sterilization

Use only CE marked and validated vacuum autoclaves. Always follow the manufacturer's instructions for use. Ensure instruments are fully clean, dried and lubricated. Use packaging materials as specified by EN 868 and ISO 11607.

Do not exceed manufacturer's stated load. Recommended temperature and time: 134°C -137°C 2.25 bar for minimum 3 minutes.

Do not exceed 140°C

3.2 Dry Heat Sterilization

Use only CE approved dry heat sterilizers.

Always follow the manufacturer's instructions for use. Recommended temperature:

160°C do not exceed 180°C

Note: Other forms of sterilization (i.e., Low temperature steam and Formaldehyde, Ethylene Oxide and Gas Plasma) are available. However, always follow the instructions for use as issued by the manufacturer and always consult them if in any doubt over the suitability of any process used.

Storage

Ensure instruments are dry before storage, and stored in dry, clean conditions at an ambient room temperature.

NOTE: IT IS THE RESPONSIBILITY OF THE PREPROCESSOR TO ENSURE THAT THE REPROCESSING AS ACTUALLY PERFORMED USING EQUIPMENT, MATERIALS AND PERSONNEL IN THE REPROCESSING FACILITY ACHIEVE THE DESIRED RESULT. THIS REQUIRES VALIDATION AND ROUTINE MONITORING OF THE PROCESS. LIKEWISE, ANY DEVIATION BY THE PREPROCESSOR FROM THE PROVIDED INSTRUCTIONS MUST BE PROPERLY EVALUATED FOR EFFECTIVENESS AND POTENTIAL ADVERSE CONSEQUENCES.

meditrans

a Proclinic Group company

e-meditrans.pl

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