(FR)

COMPOSITION Sulfato de baryum, oxyde de zinc, excipients.

PROPRIÉTÉS

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PROPERTIES

Component needed to obtain the eugenate cement.

To act as adequate root canal filling materials, pastes should be neither resorbable once hardened nor retractable over the years.

These pastes should be radiopaque and easy to insert in the canal.

ENDOMETHASONE SP has all the various above-described qualities.

PERFORMANCES

Zinc oxide of ENDOMETHASONE SP reacts with the eugenol to form eugenate cement.

INDICATIONS

Permanent root canal filling and sealing in dental procedures with systematic insertion of Gutta-Percha points.

CONTRA INDICATIONS

Hypersensitivity to one of the components

ADVERSE EVENTS

Risk of allergy to any of the constituents particularly in case of contact with gums.

INSTRUCTIONS FOR USE
The treated tooth should be isolated (using a cotton roll or rubber dam).

After carefully shaping the canal, cleaning it with an antiseptic solution and drying the canal with paper points, prepare the canal paste extemporaneously, progressively pouring the powder into the liquid (eugenol or ENDOMETHASONE Liquid) at a ratio of 2 spoonfuls of powder to 3-6 drops of liquid, until the paste has the required consistency for the selected filling technique:

- Use of a paste carrier (single cone obturation)
- And/or coating Gutta-Percha points (lateral and vertical condensation).

Then insert one or several Gutta-Percha points, which have previously been fitted to the root canal shape and size.

Take X-rays to ensure that root canals have been thoroughly filled.

Close bottle tightly after use.

Any unused paste should be discarded.

INTERACTIONS

Do not use in combination with composite materials since eugenol-based preparations inhibit their polymerization.

WARNINGS AND PRECAUTIONS FOR USE
The canal can only be sealed if it has previously been prepared, cleaned and dried.

Determine the adequate working length of the root canal before filling so as to prevent any apical extrusion, and appropriate sealing.

The spoon is provided for use only with this product. Do not reuse once the vial has been emptied. Before use, ensure the cleanliness of the spoon in accordance with good dental practices.

Danger (contains thymol iodide)
Harmful if swallowed.
Causes severe skin burns and eye damage.

In case of contact with eyes, immediately rinse the eye with plenty of fresh water for several minutes and contact a doctor.

Very toxic to aquatic life with long lasting effects.
Refer to safety data sheets (available upon request).

STORAGE

Do not store above 30°C and protect from moisture.

PRESENTATION
Bottle of 42 g powder + a spoon.

For information:
The content of the 42g bottle is estimated to ensure 86 uses of 2 full spoons

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