PL CHLORAXID 5,25%

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EN CHLORAXID 5,25%

Liquid for root canals rinsing

BEFORE USE PLEASE READ THE FOLLOWING INSTRUCTIONS CAREFULLY PRODUCT FOR DENTAL USE ONLY

COMPOSITION: sodium hypochlorite (5.25% of active chlorine), water Ph. Eur.

INTENDED USE

During the mechanical canal widening it removes the remains of dead pulp. It cleans the canal and removes the smear layer to expose the dentin root canal orifices before final filling. It prevents discoloration of the teeth that might occur after filling the non-rinsed root canal.

Adapter with a plug for taking the liquid preparations into the syringe

- Unscrew the bottle cap.
- Screw the adapter on the bottle.
- In order to use the bottle, remove the blue plug, screw the end of the syringe into the adapter opening, tilt the bottle with the preparation upside down and take the liquid into the syringe, while simultaneously holding down the adapter. Next, put the bottle upright on the table top and gently unscrew the filled syringe from adapter. It is recommended to use luer-lock syringes with a rubber plunger. Zaleca się stosowanie strzykawk luer-lock z gumowym tłokiem.
- After each use, close the adapter opening with a blue plug.
- For transport, remove the adapter and screw the cap on the bottle.

INSTRUCTIONS FOR USE

Insert the liquid CHLORAXID 5.25% into the root canal using the syringe with a needle with a side opening. Rinse the canals copiously, approximately 10 ml for one canal. Sodium hypochlorite contained in the product inactivates immediately on contact with organic substance. As the concentration of sodium hypochlorite decreases, the rinsing time and the volume of the solution used should be increased.

CONTRAINDICATIONS

Do not use the product CHLORAXID 5.25% for patients with hypersensitivity to the product's components.

CAUTION! While rinsing root canals, use physiological saline between rinsing with sodium hypochlorite and chlorhexidine digluconate. It prevents the precipitation of a brown deposit. Rinse copiously.

PRECAUTIONS

During root canal preparation do not use a too heavy stream of the liquid because it can cause the liquid to penetrate the periapical tissues and damage them, resulting in pain and swelling of soft tissues. Special care should be taken when the product is applied into perforations. May cause severe skin burns and eye damage. In case of contact rinse immediately with plenty of water and seek medical advice. In case of consumption do not induce vomiting. Drink plenty of water. Seek medical advice. Use rubber-dam while working with the product. The use of personal protective equipment (glasses, protective visor, gloves) is recommended. **DISPENSER IS NOT INTENDED FOR LIQUID STORAGE!** Any serious incident involving the product should be reported to the manufacturer and the competent authority of the State in which the user or patient is domiciled.

STORAGE

Store in original packaging at 2-8 °C (in a refrigerator). Store in a securely closed packaging. Protect from sunlight. Keep out of reach of children. Expiry date is placed on the main container. Once the container is open, the expiry date doesn't change if it the container is securely closed after each use.

TRANSPORT

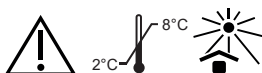
The manufacturer allows transport of the product for a period not longer than 10 days at a temperature not exceeding 40 °C. Short-term transport within the given temperature range does not affect the safety and efficacy of the product throughout its shelf life.

HANDLING THE PRODUCT CONTAINER

Empty containers should be disposed of responsibly or returned to the manufacturer

PACKAGING

Bottle of 200 g or 400 g of preparation, adapter with a screw cap, dispenser.

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