## PL Endo-Prep CREAM

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## EN Endo-Prep CREAM

Cream for root canals preparation

**BEFORE USE PLEASE READ THE FOLLOWING INSTRUCTIONS CAREFULLY**  
PRODUCT FOR DENTAL USE ONLY

**COMPOSITION:** disodium edetate (EDTA 15%), urea peroxide 10%, excipients.

### INTENDED USE

The preparation ENDO-PREP CREAM is intended for chemo-mechanical root canal preparation. EDTA in the product helps to remove calcium salts and pulp from decalcified root canals, allowing faster cleaning and shaping the canals. When used together with sodium hypochlorite, it whitens the teeth that can be discolored due to necrosis. Abundant foam created as a result of the reaction between urea peroxide and NaCl allows removing debris remaining and dentine from the prepared canal. Lubricating substances in the product facilitate the introduction of the tool into the duct and minimize the possibility of breaking the tool in the canal.

### INSTRUCTIONS FOR USE

Before the treatment, immerse the attached applicator into ethyl alcohol and then place it onto the syringe. Open the tooth cavity and rinse it with sodium hypochlorite. Insert the preparation into tooth cavity using the attached applicator or place it directly on the root canal tool. Prepare the canal mechanically while simultaneously rinsing it with sodium hypochlorite. Rinse the root canal and the cavity with sodium hypochlorite until it s/ tops foaming. A reusable product with a single-use applicator. Re-using the applicator may cause the risk of reinfection.

### CONTRAINDICATIONS

Do not use for patients who are hypersensitive to the product components. Do not leave ENDO-PREP CREAM in the canal or the tooth cavity between visits. This can lead to excessive softening of dentin.

### PRECAUTIONS

Caution is advised when preparing the canal close to the apex area, as the tool may cause perforation of the canal. May cause severe skin burns and eye damage. In case of contact rinse with plenty of water. Seek medical advice if irritation doesn't stop. If swallowed, do not induce vomiting. Drink plenty of water. Seek medical advice. Use a rubber-dam when working with the product. Use protective gloves, glasses and protective clothing.

### STORAGE

Store in original packaging between 2-15° C. Keep out of reach of children. Expiry date is placed on the main container. Once container is open, the expiry date does not change if the container is securely closed after each use.

### TRANSPORT

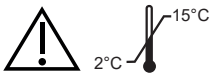
The product can be shipped for a maximum of 10 days under 40 °C. Short-distance shipping and handling within the recommended temperature range does not affect the safety and efficacy of the product throughout its shelf life.

### HANDLING THE PRODUCT CONTAINER

Empty containers should be disposed of responsibly or returned to the manufacturer.

### PACKAGING

Syringe containing 2 ml, 5 ml or 10 ml of the preparation, disposable applicators.

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